Comfort Knight Pillow Raiser

VG814

Usage and Maintenance Instructions

INTRODUCTION

Thank you for deciding to purchase the Comfort Knight™ Powered Pillow Lift from Aidapt. The Comfort Knight™ is manufactured from the finest materials and components available. When installed and used correctly it is designed to give many years of reliable trouble-free service.

Before you attempt to fit your Comfort Knight™, please take some time to read the instructions and limitations of use.

N.B. This equipment must be installed by a competent person.

140 kg (22 stone) weight limit.

Do not exceed the weight limit stated – doing so could put the user at risk.

LIMITATIONS OF USE

1. Your Comfort Knight™ is designed to rest on top of your mattress and is designed to lift one person.
2. The Comfort Knight™ is fitted to a double bed; it should be fitted to one side allowing the other occupant to remain prone when the product is in use.
3. The Comfort Knight™ can, in some instances, be unstable. It is, therefore, very important that the traversing strap and buckle provided is fitted in accordance with these instructions.
4. The Comfort Knight™ must not be used in conjunction with any form of electric under blanket for reasons of safety.
5. The Comfort Knight™ is an ideal product for home and nursing home situations. It has been tested for low to medium hazard level.
6. The Comfort Knight™ has been tested to a safe working load of 140 kg (22 stone) evenly distributed over the unit.
7. For reasons of safety you must not exceed the safe working load as stated of 140 kg (22 stone).

PRODUCT DESCRIPTION

1. **FRAME.** Your Comfort Knight™ is constructed from high quality, strong steel tube and flat metal frame. All of the frame is coated with an epoxy paint for longevity. The Comfort Knight™ is powered by a lifting device known as an actuator, which comes ready fitted to the frame. It is the actuator that gives the raising and lowering action to the Comfort Knight™.
2. **HANDSET.** Your Comfort Knight™ is ergonomically designed to feel comfortable to the hand. It has 'easy touch' buttons with clear arrow-shaped markings for raise and lower. An indicator light is included as standard, which illuminates when your Comfort Knight™ is working.
3. Each Comfort Knight™ comes complete with a webbing strap and buckle, this must be securely fitted. (Please see Fitting Instructions.) The webbing strap may come loose after a period of use; please ensure that the strap is checked periodically for tightness.
4. **CONTROL BOX AND MAIN PLUG:** Each Comfort Knight™ is provided with a separate transformer control box, which can be rested on the floor adjacent to the bed.

BEFORE USE/UNPACKING YOUR QUIET KNIGHT

Firstly please check that the box contains all the components required which are as follows:

1. The Comfort Knight™ Pillow Lift includes:
 - Frame with actuator fitted
 - Separate transformer control box and mains plug
 - Separate handset
 - 1 x webbing strap
 - Slip over foam mattress with removeable cover
 - Packing List
 - Test Certificate
 - User Instruction Manual

If you are short of any of the above items or any of the units are damaged, please contact your supplier immediately quoting the Serial No. & Packing Details, e.g., Date & Packers No.

2. Do not remove the plastic locking transit strap fitted to the rear of the Comfort Knight™ until fitted securely to the mattress.

HOW TO FIT YOUR COMFORT KNIGHT™

1. Firstly, fit the webbing strap and buckle provided to the frame of the Comfort Knight™ as illustrated. See Illustration 5.1.

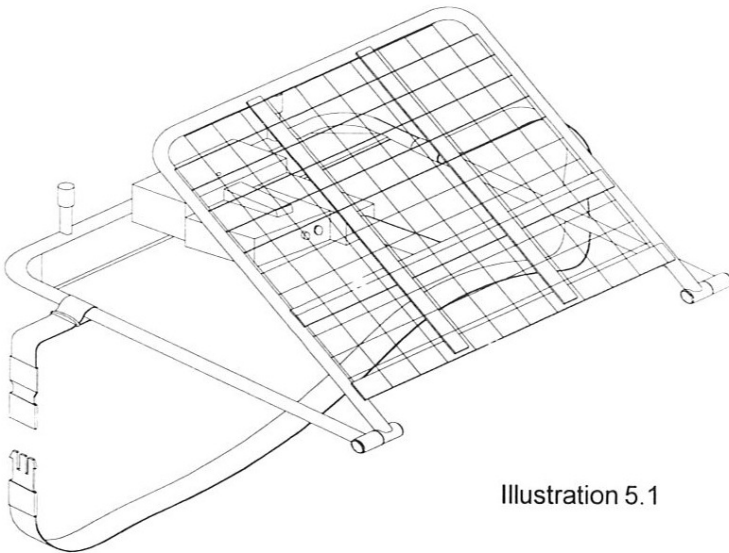
Illustration 5.1

2. Place the Comfort Knight™ on top of the mattress.

3. Pass the webbing strap laterally positioning it as far to the rear of the unit as possible across the top of the mattress, passing the strap under the bed and securing the strap to the buckle you have fitted previously. See Illustration 5.3.

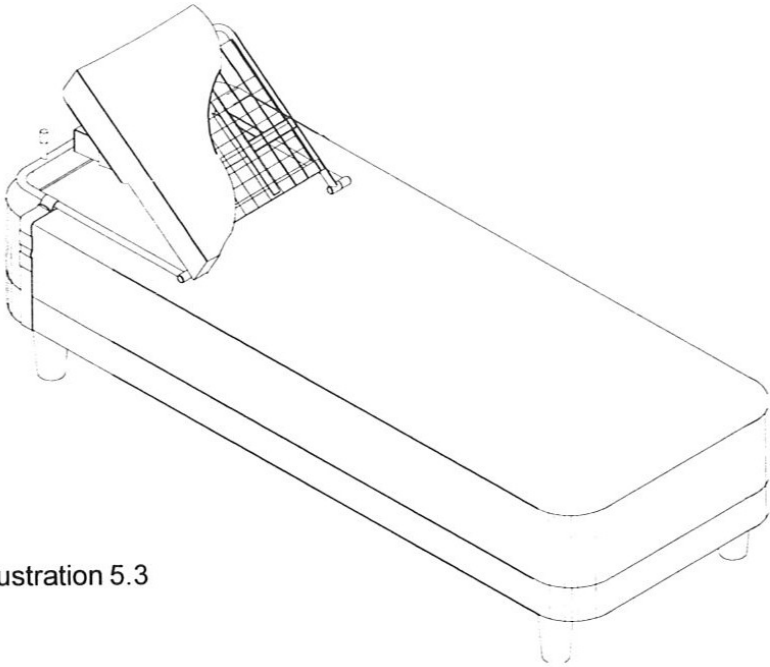
Illustration 5.3

4. Now remove the plastic transit strap fitted to the rear of the frame.
Remove the strap using scissors or a sharp knife.

Before removing the transit strap be sure to wear protective goggles.

Ensure that when using scissors or sharp instruments that you do not injure yourself or cut into any other part of the unit.

You must be sure to remove the transit strap **BEFORE** operating the Comfort Knight™.

5. Take the mattress cover provided with its kangaroo style pouch and pass the pouch over the top pivoted section of the frame.
Tie off the mattress cover around the bottom end of the pivoting section of the frame with the integral ribbon provided. See Illustration 5.5.

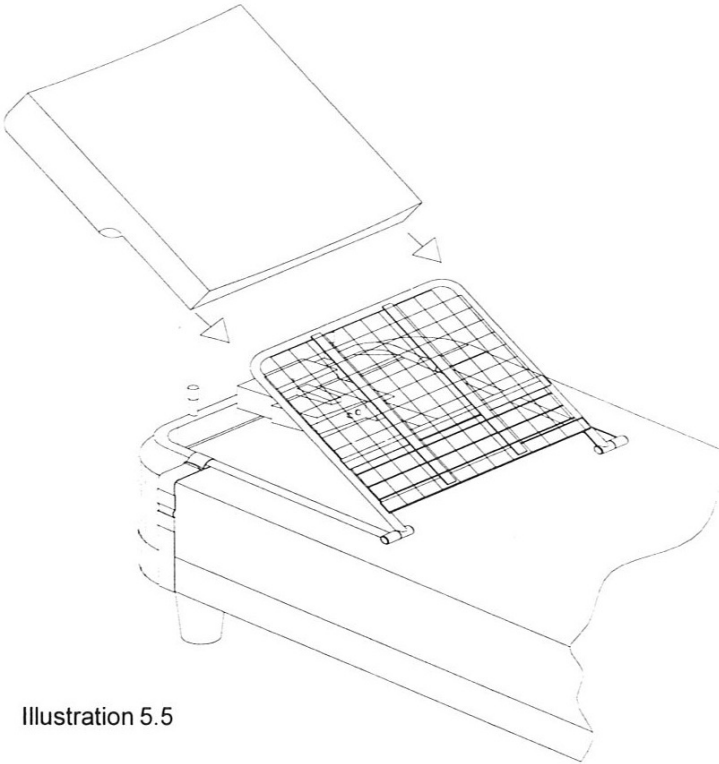
Illustration 5.5

6. Plug in the actuator to the transformer control box provided.
7. Plug in the handset provided.

8. Plug in the mains 13 amp 3 pin plug fused to 3 amps maximum provided. See Illustration.5.6.

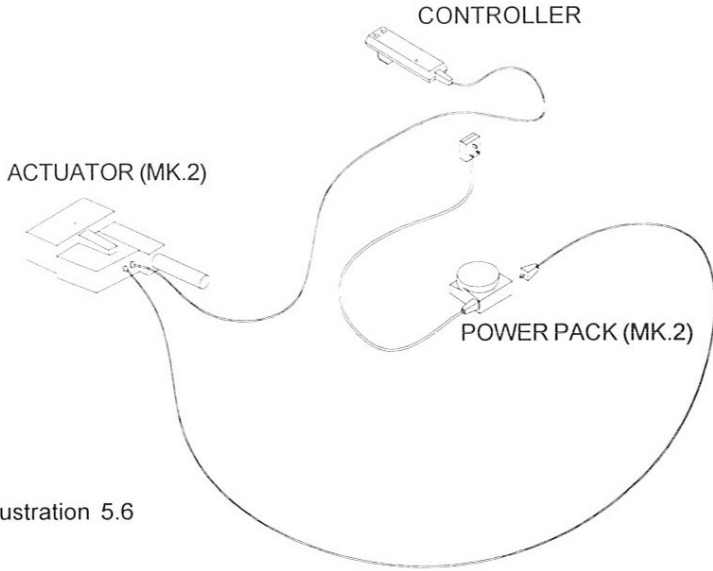
Illustration 5.6

FAULT FINDING

PROBLEM	POSSIBLE CAUSE	REMEDY
Comfort Knight™ continues to stop sporadically	Safe working load exceeded.	Follow safe load guidelines, 140 kg (22 stone).
	Duty Cycle 6 minutes continued use exceeded	Allow the unit to 'rest'. Do not exceed 6 minutes continuous use.
	Possible loose handset	Check handset connections.
	Mains cable loose.	Check mains cable.
Comfort Knight™ is omitting noise	Actuator needs running in.	Run a few times to seat actuator.
Actuator is running but the unit is not moving	Faulty actuator or handset.	Call engineer.
	Obstacle is between the base and pivoting section of the frame causing the auto spline to disengage.	Remove obstacle. The safely cut out will re-engage.
Unit not running/No Power	Mains cable not plugged in.	Check plug to wall socket.
	Handset not plugged in.	Check connection.
	Fuse to the mains plug has blown.	Check fuse and replace with 3 amp max.
	Electricity supply interrupted.	Check fuse board.
Unit wobbles in service	Strap not fitted.	Fit strap.
	Strap is loose.	Tighten strap.

REISSUE

If you reissue or are about to reissue this product, please thoroughly check all components for their safety.

If in any doubt, please do not issue or use, but immediately contact your supplier for service support.

CARE & MAINTENANCE

1. The Comfort Knight™ must only be serviced by a trained, competent engineer appointed by Aidapt Bathrooms Ltd.
2. Regularly check that the lateral strap is secure.
3. When cleaning the Comfort Knight™ make sure you completely cover the actuator to avoid any liquid ingress and remove all connections to mains and the actuator itself.
4. Always remove the Comfort Knight™ mattress and cover from the unit and clean separately.
5. Only use soap and water or disinfectant to BS 6424 to clean all parts of the Comfort Knight™. Excluding the actuator and other electrical parts.
6. Should you have a blown fuse to the 3 pin main plug, ensure that it is replaced only with a 3 Amp fuse.

IMPORTANT INFORMATION

The information given in this instruction booklet must not be taken as forming part of or establishing any contractual or other commitment by Aidapt Bathrooms Limited, Aidapt (Wales) Ltd or its agents or its subsidiaries and no warranty or representation concerning the information is given.

Please exercise common sense and do not take any unnecessary risks when using this product; as the user you must accept liability for safety when using the product.

Please do not hesitate to contact the person who issued this product to you or the manufacturer (detailed below) if you have any queries concerning the assembly/use of your product.

Aidapt Bathrooms Ltd, Lancots Lane, Sutton Oak, St Helens, WA9 3EX
Telephone: +44 (0) 1744 745 020 • Fax: +44 (0) 1744 745 001 • Web: www.aidapt.co.uk
Email: accounts@aidapt.co.uk • adaptations@aidapt.co.uk • sales@aidapt.co.uk

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Altai France (Altai Europe Ltd)