## Grip-On Chair Raisers

## Usage and Maintenance Instructions

## INTRODUCTION

Thank you for deciding to purchase the Grip-on Chair Raisers from Aidapt. The Grip-on Chair Raisers are manufactured from the finest materials available. When used correctly they are designed to give many years of reliable, trouble-free service.

### **NB. This equipment must be installed by a competent person.**

VG823D Max. Diameter: 32 mm (1 1/4")

VG823F Max. Diameter: 45 mm (1 3/4")

VG823H Max. Diameter: 58 mm (2 1/4")

Maximum User Weight: 500 kg (78 stone) including furniture.

**Do not exceed the weight limit stated – doing so could put the user at risk.**

## BEFORE USE

Carefully remove all packaging. Avoid using any knives or other sharp instruments as this may damage the surface of the product.

Check the product for any visible damage. If you see any damage or suspect a fault, do not use your product, but contact your supplier for support.

## INTENDED USE

Grip-on Chair Raisers are suitable for use with furniture on castors or legs.

The Grip-on Chair Raisers grip onto the furniture using the unique 'donut'. This greatly increases safety as the furniture raiser is attached to the furniture rather than the furniture simply sitting on the raiser, as with some other models.

No additional fixing is required due to the gripping nature of the 'donut' in each Chair Raiser.

Raisers are supplied in a set of 4.

To raise 25 mm (1") one set is required, 2 sets are required to raise 50 mm (2") and 3 sets are required to raise 75 mm (3").

Available to fit several widths of castors/legs.

## **FITTING YOUR CHAIR RAISERS**

### **NB. Two people may be required to fit the Chair Raisers safely.**

Lift the furniture and insert each leg into the central 'donut' of the Grip-on Chair Raiser.

Carefully lower the furniture to the ground, checking that the leg/castor is securely fitted into the 'donut' and that the base of the Chair Raiser is flat and level on the ground before use.

## **CLEANING**

Clean your Grip-on Chair Raisers using a mild detergent with a slightly damp, soft cloth only. Only use a mild detergent or disinfectant; never use an abrasive cleaner.

After cleaning, wipe the raisers with a dry, soft cloth if necessary. Do not leave in direct contact with a heat source.

## **REISSUE**

If you reissue or are about to reissue this product, please thoroughly check all components for their safety.

**If in any doubt, please do not issue or use, but immediately contact your supplier for service support.**

## **CARE & MAINTENANCE**

Please make a safety check of the product at regular intervals or if you have any concerns.

## **IMPORTANT INFORMATION**

The information given in this instruction booklet must not be taken as forming part of or establishing any contractual or other commitment by Aidapt Bathrooms Limited, Aidapt (Wales) Ltd or its agents or its subsidiaries and no warranty or representation concerning the information is given.

Please exercise common sense and do not take any unnecessary risks when using this product; as the user you must accept liability for safety when using the product.

Please do not hesitate to contact the person who issued this product to you or the manufacturer (detailed below) if you have any queries concerning the assembly/use of your product.

**Aidapt Bathrooms Ltd, Lancots Lane, Sutton Oak, St Helens, WA9 3EX**  
**Telephone: +44 (0) 1744 745 020 • Fax: +44 (0) 1744 745 001 • Web: [www.aidapt.co.uk](http://www.aidapt.co.uk)**  
**Email: [accounts@aidapt.co.uk](mailto:accounts@aidapt.co.uk) • [adaptations@aidapt.co.uk](mailto:adaptations@aidapt.co.uk) • [sales@aidapt.co.uk](mailto:sales@aidapt.co.uk)**

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**Altai Germany (Altai Europe Ltd.)**